



August 17, 2020

Amy Zhang, Ph.D.
VP and General Manager
BioCheck, Inc.
425 Eccles Avenue
South San Francisco, CA 94080

Device: BioCheck SARS-CoV-2 IgG and IgM Combo Test

Company: BioCheck, Inc.

Indication: Qualitative detection and differentiation of IgG and IgM antibodies to SARS-CoV-2 in human serum using the MS-Fast Automated Chemiluminescent Immunoassay Analyzing System. Intended for use as an aid in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating recent or prior infection. Emergency use of this test is limited to authorized laboratories.

Authorized Laboratories: Laboratories certified under the Clinical Laboratory Improvement Amendments of 1988 (CLIA), 42 U.S.C. 263a, that meet requirements to perform moderate or high complexity tests.

Dear Dr. Zhang:

This letter is in response to your¹ request that the Food and Drug Administration (FDA) issue an Emergency Use Authorization (EUA) for emergency use of your product,² pursuant to Section 564 of the Federal Food, Drug, and Cosmetic Act (the Act) (21 U.S.C. §360bbb-3).

On February 4, 2020, pursuant to Section 564(b)(1)(C) of the Act, the Secretary of the Department of Health and Human Services (HHS) determined that there is a public health emergency that has a significant potential to affect national security or the health and security of United States citizens living abroad, and that involves the virus that causes COVID-19. Pursuant to Section 564 of the Act, and on the basis of such determination, the Secretary of HHS then declared that circumstances exist justifying the authorization of emergency use of in

¹ For ease of reference, this letter will use the term “you” and related terms to refer to BioCheck, Inc.

² For ease of reference, this letter will use the term “your product” to refer to the BioCheck SARS-CoV-2 IgG and IgM Combo Test for the indication identified above.

vitro diagnostics for detection and/or diagnosis of the virus that causes COVID-19 subject to the terms of any authorization issued under Section 564(a) of the Act.³

Having concluded that the criteria for issuance of this authorization under Section 564(c) of the Act are met, I am authorizing the emergency use of your product, described in the Scope of Authorization of this letter (Section II), subject to the terms of this authorization.

I. Criteria for Issuance of Authorization

I have concluded that the emergency use of your product meets the criteria for issuance of an authorization under Section 564(c) of the Act, because I have concluded that:

1. The SARS-CoV-2 can cause a serious or life-threatening disease or condition, including severe respiratory illness, to humans infected by this virus;
2. Based on the totality of scientific evidence available to FDA, it is reasonable to believe that your product may be effective in diagnosing recent or prior infection with SARS-CoV-2 by identifying individuals with an adaptive immune response to the virus that causes COVID-19, and that the known and potential benefits of your product when used for such use, outweigh the known and potential risks of your product; and
3. There is no adequate, approved, and available alternative to the emergency use of your product.⁴

II. Scope of Authorization

I have concluded, pursuant to Section 564(d)(1) of the Act, that the scope of this authorization is limited to the indication above.

Authorized Product Details

Your product is a qualitative test intended for the detection and differentiation of IgG and IgM antibodies to SARS-CoV-2 in human serum using the MS-Fast Automated Chemiluminescent Immunoassay Analyzing System. The product is intended for use as an aid in identifying individuals with an adaptive immune response to SARS-CoV-2, indicating recent or prior infection. At this time, it is unknown for how long antibodies persist following infection and if the presence of antibodies confers protective immunity.

The product uses magnetic separation technology in two separate reactions to detect SARS-CoV-2 IgG and IgM antibodies with magnetic particles as the solid phase of the reaction. Serum specimen, biotinylated SARS-CoV-2 antigen, and streptavidin coated magnetic beads are mixed together during incubation at 37°C for 10 minutes. Material that is not bound to the streptavidin coated magnetic beads is removed by washing. Alkaline phosphatase labeled anti-human IgG is

³ U.S. Department of Health and Human Services, *Determination of a Public Health Emergency and Declaration that Circumstances Exist Justifying Authorizations Pursuant to Section 564(b) of the Federal Food, Drug, and Cosmetic Act*, 21 U.S.C. § 360bbb-3. 85 FR 7316 (February 7, 2020).

⁴ No other criteria of issuance have been prescribed by regulation under Section 564(c)(4) of the Act.

then added to one of the complexes and alkaline phosphatase labeled anti-human IgM is added to the other. Following another wash to remove excess alkaline phosphatase labeled anti-human IgG and IgM antibodies, luminescent substrate is added and the enzyme in the complex catalyzes the luminescent substrate to form an unstable excited intermediate. When the excited intermediate returns to the ground state, it emits a photon. The number of photons produced is positively correlated with the content of SARS-CoV-2 IgG and IgM antibodies in the sample. The photons emitted is detected by a photomultiplier tube in the MS-Fast and the content of SARS-CoV-2 IgG/IgM antibodies in the sample will be expressed as Relative Light Unit (RLU) and results will be calculated and displayed by MS-Fast. Samples that are positive for SARS CoV-2 IgM will have $RLU \geq 18,500$ RLU whereas samples that are negative will have $RLU < 18,500$. Samples that are positive for SARS CoV-2 IgG will have $RLU \geq 26,000$ RLU whereas samples that are negative will have $RLU < 18,500$.

Your product requires the following authorized control materials which are provided with your product or other authorized control materials (refer to Condition P. below):

- Positive Control IgM: BSA-containing buffer with human SARS-CoV-2 S1 IgM antibody
- Negative Control IgM: BSA-containing buffer
- Positive Control IgG: BSA-containing buffer with human SARS-CoV-2 S1 IgG antibody
- Negative Control IgG: BSA-containing buffer.

Your product also requires the use of additional authorized materials and authorized ancillary reagents that are not included with your product and are described in the Instructions for Use.

Your above described product is authorized to be accompanied with labeling entitled “BioCheck SARS-CoV-2 IgG and IgM Antibody Combo Test Kit Instructions for Use” and the “MS-Fast Automated Chemiluminescent Immunoassay Analyzing System Manual” (available at <https://www.fda.gov/medical-devices/emergency-situations-medical-devices/emergency-use-authorizations>), and the following product-specific information pertaining to the emergency use, which is required to be made available to healthcare providers and recipients:

- Fact Sheet for Healthcare Providers: BioCheck Inc.- BioCheck SARS-CoV-2 IgG and IgM Combo Test
- Fact Sheet for Recipients: BioCheck Inc.- BioCheck SARS-CoV-2 IgG and IgM Combo Test

The above described product, when accompanied by the Instructions for Use (identified above) and the two Fact Sheets (collectively referenced as “authorized labeling”) is authorized to be distributed to and used by authorized laboratories under this EUA, despite the fact that it does not meet certain requirements otherwise required by applicable federal law.

I have concluded, pursuant to Section 564(d)(2) of the Act, that it is reasonable to believe that the known and potential benefits of your product, when used consistent with the Scope of Authorization of this letter (Section II), outweigh the known and potential risks of your product.

I have concluded, pursuant to Section 564(d)(3) of the Act, based on the totality of scientific evidence available to FDA, that it is reasonable to believe that your product may be effective in diagnosing recent or prior infection with SARS-CoV-2 by identifying individuals with an adaptive immune response to the virus that causes COVID-19, when used consistent with the Scope of Authorization of this letter (Section II), pursuant to Section 564(c)(2)(A) of the Act.

FDA has reviewed the scientific information available to FDA, including the information supporting the conclusions described in Section I above, and concludes that your product (as described in the Scope of Authorization of this letter (Section II) meets the criteria set forth in Section 564(c) of the Act concerning safety and potential effectiveness.

The emergency use of your product under this EUA must be consistent with, and may not exceed, the terms of this letter, including the Scope of Authorization (Section II) and the Conditions of Authorization (Section IV). Subject to the terms of this EUA and under the circumstances set forth in the Secretary of HHS's determination under Section 564(b)(1)(C) of the Act described above and the Secretary of HHS's corresponding declaration under Section 564(b)(1) of the Act, your product is authorized for the indication above.

III. Waiver of Certain Requirements

I am waiving the following requirements for your product during the duration of this EUA:

- Current good manufacturing practice requirements, including the quality system requirements under 21 CFR Part 820 with respect to the design, manufacture, packaging, labeling, storage, and distribution of your product, but excluding Subpart H (Acceptance Activities, 21 CFR 820.80 and 21 CFR 820.86), Subpart I (Nonconforming Product, 21 CFR 820.90), and Subpart O (Statistical Techniques, 21 CFR 820.250).

IV. Conditions of Authorization

Pursuant to Section 564(e) of the Act, I am establishing the following conditions on this authorization:

BioCheck, Inc. (You) and Authorized Distributor(s)⁵

- A. Your product must comply with the following labeling requirements under FDA regulations: the intended use statement (21 CFR 809.10(a)(2), (b)(2)); adequate directions for use (21 U.S.C. 352(f)), (21 CFR 809.10(b)(5), (7), and (8)); appropriate limitations on the use of the device including information required under 21 CFR

⁵ “Authorized Distributor(s)” are identified by you, BioCheck, Inc., in your EUA submission as an entity allowed to distribute your device.

809.10(a)(4); and any available information regarding performance of the device, including requirements under 21 CFR 809.10(b)(12).

- B. You and authorized distributor(s) will make your product available with the authorized labeling to authorized laboratories. You may request changes to the authorized labeling. Such requests will be made in consultation with, and require concurrence of, Division of Microbiology Devices/Office of Health Technology-7 – Office of In Vitro Diagnostics and Radiological Health/ Office of Product Evaluation and Quality/ Center for Devices and Radiological Health (DMD/OHT7-OIR/OPEQ/CDRH).
- C. You and authorized distributor(s) will make available on your website(s) the Fact Sheet for Healthcare Providers and the Fact Sheet for Recipients.
- D. You and authorized distributor(s) will inform authorized laboratories and relevant public health authorities of this EUA, including the terms and conditions herein, and any updates made to your product and authorized labeling.
- E. Through a process of inventory control, you and authorized distributor(s) will maintain records of the authorized laboratories to which they distribute the test and number of tests they distribute.
- F. You and authorized distributor(s) will collect information on the performance of your product. You will report to FDA any suspected occurrence of false positive and false negative results and significant deviations from the established performance characteristics of the product of which you become aware.
- G. You and authorized distributor(s) are authorized to make available additional information relating to the emergency use of your product that is consistent with, and does not exceed, the terms of this letter of authorization.

BioCheck, Inc. (You)

- H. You will notify FDA of any authorized distributor(s) of your product, including the name, address, and phone number of any authorized distributor(s).
- I. You will provide authorized distributor(s) with a copy of this EUA and communicate to authorized distributor(s) any subsequent amendments that might be made to this EUA and its authorized accompanying materials (e.g., Fact Sheets).
- J. You may request to make available additional authorized labeling specific to an authorized distributor. Such additional labeling may use another name for the product, but otherwise must be consistent with the authorized labeling, and not exceed the terms of authorization of this letter. Such requests will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.

- K. You will comply with the following requirements under FDA regulations: Subpart H (Acceptance Activities, 21 CFR 820.80 and 21 CFR 820.86), Subpart I (Nonconforming Product, CFR 820.90), and Subpart O (Statistical Techniques, 21 CFR 820.250).
- L. You may request changes to the Scope of Authorization (Section II in this letter) of your product. Such requests will be made in consultation with DMD/OHT7-OIR/OPEQ/CDRH, and require concurrence of, Office of Counterterrorism and Emerging Threats (OCET)/Office of the Chief Scientist (OCS)/Office of the Commissioner (OC) and DMD/OHT7-OIR/OPEQ/CDRH.
- M. You may request the addition of other instruments and associated software for use with your product. Such requests will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- N. You may request the addition of other ancillary methods for use with your product. Such requests will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- O. You may request the addition of other specimen types for use with your product. Such requests will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- P. You may request the addition and/or substitution of control materials for use with your product. Such requests will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- Q. You may request substitution for or changes to the authorized materials used in the detection process of human antibodies against SARS-CoV-2. Such requests will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- R. You will evaluate the performance and assess traceability⁶ of your product with any FDA-recommended reference material(s) or established panel(s) of characterized clinical specimens. After submission to FDA and DMD/OHT7-OIR/OPEQ/CDRH's review of and concurrence with the data, you will update your labeling to reflect the additional testing. Such labeling updates will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- S. If requested by FDA, you will participate in a National Cancer Institute (NCI) study on the evaluation of your product. After submission to FDA and DMD/OHT7-OIR/OPEQ/CDRH's review of and concurrence with the data, you will update your product labeling to reflect the additional testing. Such labeling updates will be made in consultation with, and require concurrence of, DMD/OHT7-OIR/OPEQ/CDRH.
- T. You will track adverse events, including any occurrence of false results and report to FDA pursuant to 21 CFR Part 803.

⁶ Traceability refers to tracing analytical sensitivity/reactivity back to an FDA-recommended reference material.

- U. You must have lot release procedures and the lot release procedures, including the study design and statistical power, must ensure that the tests released for distribution have the clinical and analytical performance claimed in the authorized labeling.
- V. If requested by FDA, you must submit lot release procedures to FDA within 48 hours of such request, including sampling protocols, testing protocols, and acceptance criteria, that you use to release lots of your product for distribution in the US.
- W. If requested by FDA, you will periodically submit new lots for testing at NCI, or by another government agency designated by FDA, to confirm continued performance characteristics across lots. In addition, FDA may request records regarding lot release data for tests to be distributed or already distributed. If such lot release data are requested by FDA, you must provide it within 48 hours of the request.
- X. You will complete the agreed upon real-time stability study for your product. After submission to FDA and DMD/OHT7-OIR/OPEQ/CDRH's review of and concurrence with the data, you will update your product labeling to reflect the additional testing. Such labeling updates will be made in consultation with, and require concurrence of, DMD/OHT7- OIR/OPEQ/CDRH.

Authorized Laboratories

- Y. Authorized laboratories using your product will include with test result reports, all authorized Fact Sheets. Under exigent circumstances, other appropriate methods for disseminating these Fact Sheets may be used, which may include mass media.
- Z. Authorized laboratories will use your product as outlined in the authorized labeling. Deviations from the authorized procedures, including the authorized clinical specimen types, authorized control materials, authorized other ancillary reagents and authorized materials required to use your product are not permitted.
- AA. Authorized laboratories that receive your product will notify the relevant public health authorities of their intent to run your product prior to initiating testing.
- BB. Authorized laboratories using your product will have a process in place for reporting test results to healthcare providers and relevant public health authorities, as appropriate.
- CC. Authorized laboratories will collect information on the performance of your product and report to DMD/OHT7-OIR/OPEQ/CDRH (via email: CDRH-EUA-Reporting@fda.hhs.gov) and you (<http://www.biocheckinc.com/>) any suspected occurrence of false positive or false negative results and significant deviations from the established performance characteristics of your product of which they become aware.
- DD. All laboratory personnel using your product must be appropriately trained in automated immunoassay techniques and use appropriate laboratory and personal

protective equipment when handling this kit, and use your product in accordance with the authorized labeling. All laboratory personnel using the assay must also be trained in and be familiar with the interpretation of results of the product.

BioCheck, Inc. (You), Authorized Distributors and Authorized Laboratories

EE. You, authorized distributors, and authorized laboratories using your product will ensure that any records associated with this EUA are maintained until otherwise notified by FDA. Such records will be made available to FDA for inspection upon request.

Conditions Related to Printed Materials, Advertising and Promotion

FF. All descriptive printed matter, including advertising and promotional materials, relating to the use of your product, shall be consistent with the authorized labeling, as well as the terms set forth in this EUA and the applicable requirements set forth in the Act and FDA regulations.

GG. No descriptive printed matter, including advertising or promotional materials, relating to the use of your product, may represent or suggest that this test is safe or effective for the detection of SARS-CoV-2.

HH. All descriptive printed matter, including advertising and promotional materials, relating to the use of your product, shall clearly and conspicuously state that:

- This test has not been FDA cleared or approved;
- This test has been authorized by FDA under an EUA for use by authorized laboratories;
- This test has been authorized only for the presence of IgG and IgM antibodies against SARS-CoV-2, not for any other viruses or pathogens; and
- This test is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostic tests for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.

The emergency use of your product as described in this letter of authorization must comply with the conditions and all other terms of this authorization.

V. Duration of Authorization

This EUA will be effective until the declaration that circumstances exist justifying the authorization of the emergency use of in vitro diagnostic tests for detection and/or diagnosis of

COVID-19 is terminated under Section 564(b)(2) of the Act or the EUA is revoked under Section 564(g) of the Act.

Sincerely,

RADM Denise M. Hinton
Chief Scientist
Food and Drug Administration

Enclosure